
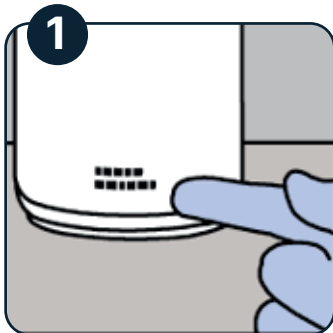


### Directions for Use

The usage directions are intended for use for daily (routine) or discharge (terminal) cleaning and disinfection protocols. Consult specific protocols for additional directions. For heavily soiled areas, pre-cleaning is necessary.

**Always check expiration dates before using.**

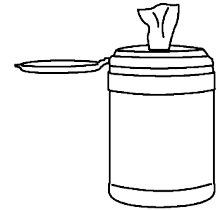


1 Check the expiration date on the back of the wipe canister. Discard if expired and get another canister.



2 Pull lid off the canister and remove seal.



3 Find and pull up the first wipe from the middle of the roll.

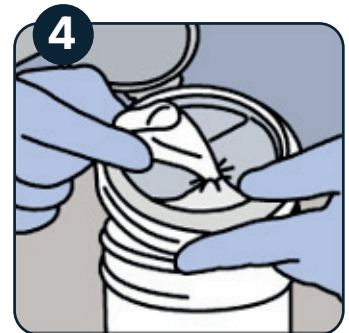

4 Thread through the opening in the center of the lid. Secure the lid.



5 Remove a wipe from the canister. Do not fold the wipe.



6 Clean and disinfect hard, non-porous surfaces or equipment surfaces.



7 Keep surfaces wet for required dwell time to achieve bactericidal and virucidal disinfection.



8 Dispose of the wipes in accordance with appropriate protocol. **DO NOT FLUSH WIPES IN TOILETS.**



9 When finished, close the canister securely so wipes will not dry out.

### Change to a new wipe...



...when the wipe does not deposit liquid on the surface.



...when the wipe is visibly soiled.



...when moving to new areas - i.e. patient room to bathroom, new patient room, or new public area.